Instruction Manual

Model HM-30, HM-30-OB

Thank you for
purchasing a
Prestige Medical
HealthMate
Blood Pressure Monitor

Prestige Medical is renowned for supplying precision medical products while delivering the highest level of service to the healthcare professional.

Your Prestige Medical blood pressure monitor has been subjected to the strictest of standards in manufacturing and quality assurance testing. With proper care and maintenance, your blood pressure monitor will provide you with years of dependable service.

Please read the following instructions prior to use.

TABLE OF CONTENTS

About blood pressure.....	3
Important information before use.....	5
Device description.....	6
Main feature.....	7
Changing batteries.....	9
Attaching arm cuff.....	9
How to measure.....	11
Maintenance.....	14
Safe-keeping.....	14
To prevent malfunction.....	14
Warning.....	15
Warranty and Service.....	15
Description of display symbols.....	16
Specifications	17

MEDICAL DISCLAIMER

This manual and product are not meant to be a substitute for advice provided by your doctor or other medical professional.

You are not to use the information contained herein, or this product for diagnosing or treating a health problem or prescribing any medication. If you have or suspect that you have a medical problem, promptly consult your healthcare provider.

INTENDED USE

Measure a person's Systolic blood pressure, Diastolic blood pressure and heart pulse on the upper arm by an oscillometric measurement method.

The product is recommended for use by people over the age of 18 and is designed for home use only.

ABOUT BLOOD PRESSURE

1. What is blood pressure?

Blood pressure is the pressure or the force of blood flowing against the walls of the arteries. Blood pressure is constantly changing during the course of the cardiac cycle. Systolic blood pressure is the highest pressure in the cycle. Diastolic blood pressure is the lowest pressure in the cycle. Both pressure readings, the Systolic rate and the Diastolic rate, are information for the physician to evaluate the status of patient's blood pressure. Many factors such as physical activities, anxieties and time of day can influence one's blood pressure. Blood pressure is typically low in the morning and increases from the afternoon to the evening. It is lower in the summer and higher in the winter.

2. Why is it useful to measure blood pressure at home?

Having one's blood pressure measured by a doctor in a hospital, or clinic tends to stimulate nervousness in a person and may cause the blood pressure to rise.


Blood pressure changes in different conditions, so judgment on the basis of a single measurement is not reliable.

Blood pressure measured first thing in the morning after waking up and before taking any food is known as the fundamental blood pressure. This is why it is useful to take blood pressure at home.

ABOUT BLOOD PRESSURE

New W.H.O blood pressure classifications

Standards for assessing high or low blood pressure without regard to age have been established by the World Health Organization (WHO) as shown in the chart to the right.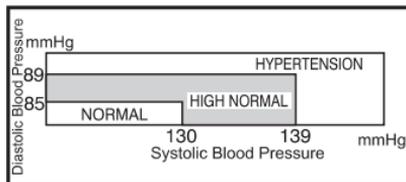


Variations in blood pressure

An individual's blood pressure varies greatly on a daily and seasonal basis. These variations are even more pronounced in hypertension patients. Normally the blood pressure rises while at work and is at its lowest during sleep.

***Note**

Hypertension: means a person who has high blood pressure symptoms.

The graph to the right illustrates the variations in blood pressure over a whole day with measurement taken every five minutes.

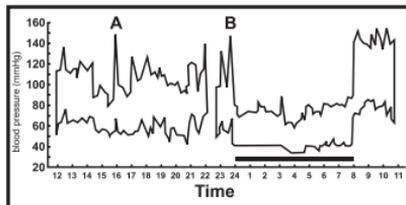
The thick line represents sleep. The rise in blood pressure at 4 PM (A in the graph) and 12 PM (B in the graph) correspond to an attack of pain.

IMPORTANT INFORMATION BEFORE USE

1. Blood pressure measurements should be interpreted by a physician or trained health professional who is familiar with your medical history. By using the unit regularly and recording the results for a physician to interpret, you can keep a physician informed of the continuing trends in your blood pressure.
2. Wrap the cuff snugly around your arm, the cuff must be at the same level as your heart.
3. Do not move the unit during measurement or an accurate reading will not be achieved.
4. Take measurement in a relaxed position without talking.
5. Do not wrap the cuff over a jacket/sweater sleeve or an accurate reading will not be achieved.
6. Keep in mind that blood pressure naturally varies throughout the day and is affected by many different factors such as smoking, alcohol consumption, medication and physical activity.
7. Blood pressure measurements taken with the unit are equivalent to those obtained by a trained observer using the cuff/stethoscope auscultation method and are within the accuracy limits prescribed by the American National Standard for Electronic or Automated Sphygmomanometers.

DEVICE DESCRIPTION


A. Names of the parts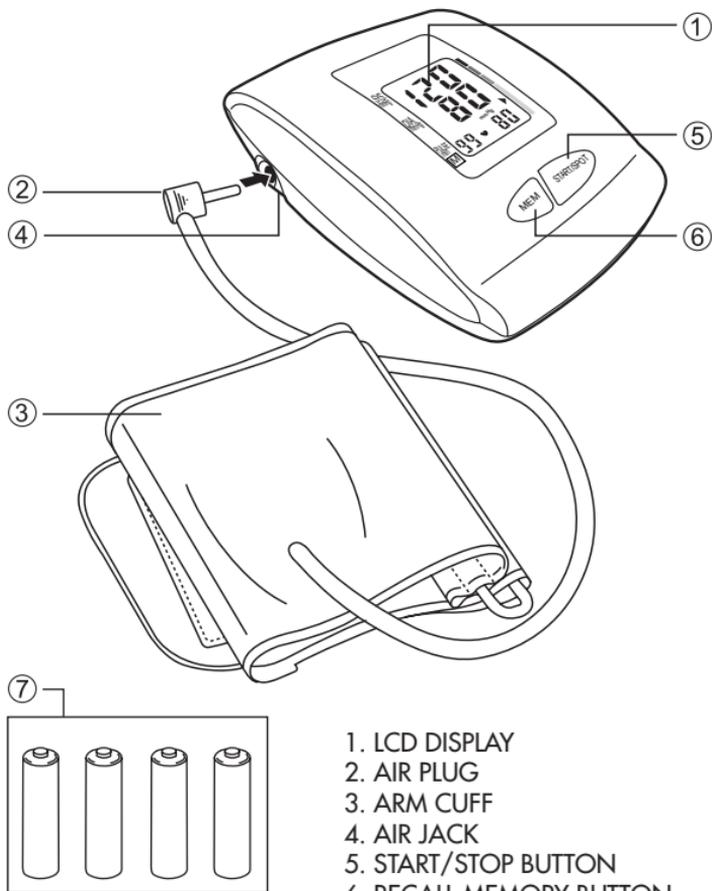

1. LCD DISPLAY
2. AIR PLUG
3. ARM CUFF
4. AIR JACK
5. START/STOP BUTTON
6. RECALL MEMORY BUTTON
7. (AA) 1.5 V x 4 ALKALINE BATTERIES

DEVICE DESCRIPTION

B. Description of display symbols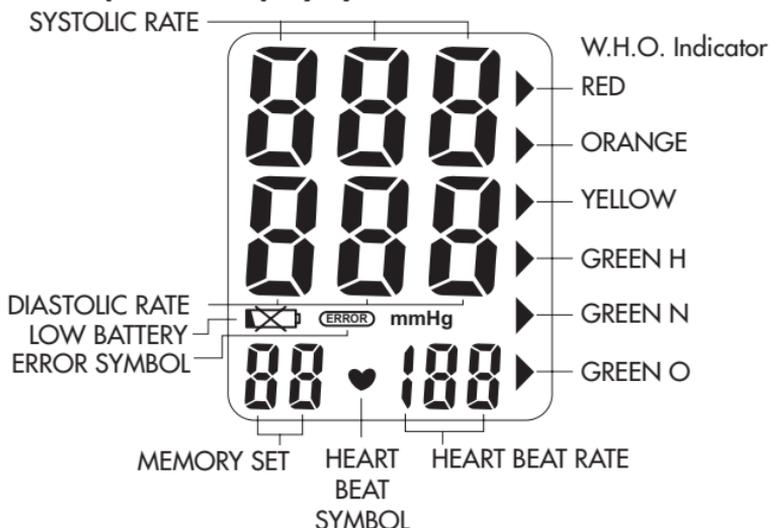

MAIN FEATURE

WHO Indicator


This device is equipped with a WHO Blood Pressure Indicator which classifies your blood pressure measurements into six stages (Optimal to Severe hypertension) based on a WHO classification on blood pressure levels as shown in below chart:

Stages of blood pressure levels	Systolic(in mmHg)	Diastolic(in mmHg)
Grade 3 Severe hypertension	≥ 180	≥ 110
Grade 2 Moderate hypertension	160~179	100~109
Grade 1 Mild hypertension	140~159	90~99
High-normal	130~139	85~89
Normal	120~129	80~84
Optimal	< 120	< 80

*Source: WHO, 1999

MAIN FEATURE

After each measurement is completed, the LCD display will show your position automatically on the six segments of the bar indicator which corresponds to the WHO Blood Pressure Indicator.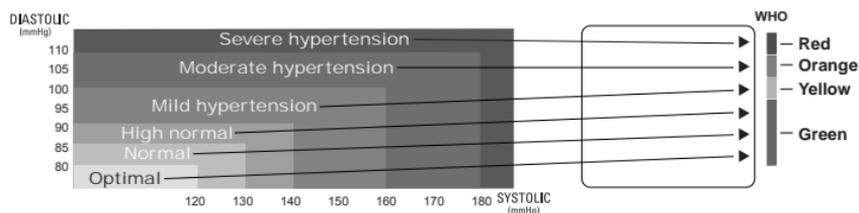

*Note!

When a person's systolic and diastolic pressures fall into different categories, the higher category should apply.

e.g. systolic rate 181 & diastolic rate 99

- the result should be in Red category (Severe hypertension)

systolic rate 110 & diastolic rate 95

- Yellow category (Mild hypertension)


CHANGING BATTERIES


Remove the battery cover at the rear side of the unit and insert ALKALINE batteries into the battery compartment as shown. Make sure the polarities + and - are observed.

Important notice :

1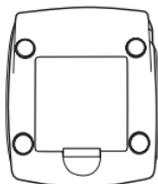

2

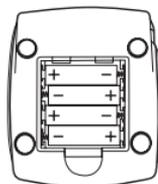
***Attention!**

Batteries are hazardous waste.


Do not dispose of it together with the household garbage.

ATTACHING ARM CUFF

1. Press your brachial artery with two fingers approximately 1 inch above the elbow on the inside of your left arm.

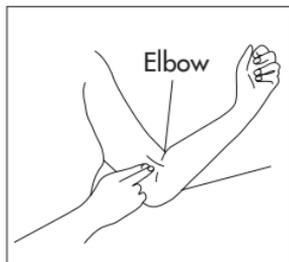



2. Determine where your strongest pulse is. Slide the end of the arm cuff furthest from the tube through the metal ring to the loop. The smooth cloth should be on the inside of the cuff.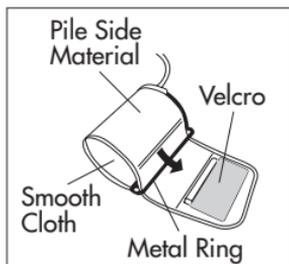

ATTACHING ARM CUFF


- If the cuff is positioned properly, the velcro will be on the outside of the cuff and the metal ring will not touch your skin.
- Put your left arm through the cuff loop. The bottom of the cuff should be about half an inch above your elbow. The cuff should lie over the brachial artery on the inside of the arm.
- Pull the cuff so that the top and bottom edges are tightened around your arm.
- When the cuff is positioned properly, press the velcro firmly against the pile side of the cuff.
- Sit in a chair and place your arm on the table so that the cuff is at the same level as your heart.
- Relax your arm and turn your arm upward.
- Make sure there are no kinks in the air tube.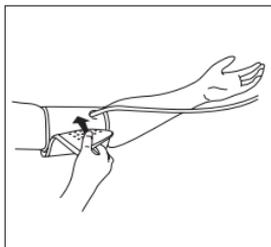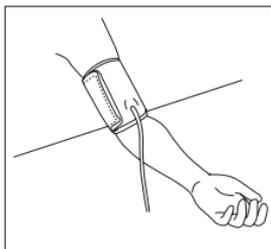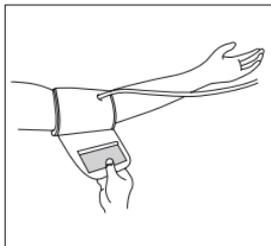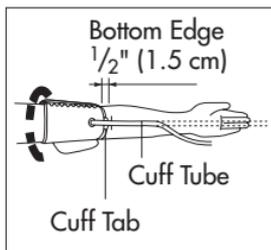

HOW TO MEASURE

1. Insert 4 "AA" Alkaline batteries in position:

- Open battery cover
- Refer to the picture in positioning batteries.
- All segments will appear on the display in 3 seconds.
- Device is ready to start.

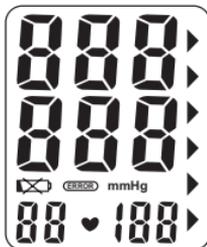2. Steps to take blood pressure measurement:

- Wrap the cuff around the arm (Refer to pages 9 and 10).
- Sit upright on the chair to have correct posture. (Refer to pages 9 and 10)
- Press START/STOP button, all segments will appear on the display in 3 seconds. Then, the monitor will automatically inflate to the level that is right for you.

Display indicates the air pressure is currently inflating.

***Note!**

Do not move or talk during taking blood pressure measurement.


- After the air pressure has increased, it will slowly decrease. When the pulse rate is detected, the Heart Beat Symbol  will start flashing.

Display indicates the air pressure is deflating and the pulse is detected.

HOW TO MEASURE

- e. After taking the blood pressure measurement, the systolic rate, diastolic rate and pulse rate will be on the display for one minute. The unit will auto shut off if a button is not pressed for one minute.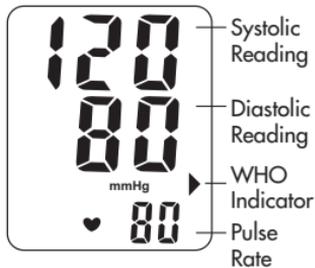

***Note!**

To stop measuring or exit any mode in process, press "START/STOP" button.

3. Storing, recalling and erasing measurement data:

a. Storing data:

After each blood pressure measurement, the systolic pressure, diastolic pressure, pulse rate will be automatically stored. The memory holds the latest 99 measurements. After 99 measurements are stored, the memory will automatically clear out the earliest data.

HOW TO MEASURE

b. Recalling data:

- (1) Press MEM button. No reading will appear on the display if there is no data in the memory. If there is data in the memory, the latest recorded sets of measurements (systolic pressure, diastolic pressure, and pulse rate) will appear on the display for one minute.
- (2) Press MEM button to read the next line of stored data. The data called up on the display is numbered accordingly.
- (3) To stop reading from the memory, press START/STOP button.

c. Erasing data:

- (1) Take out the batteries and then put them back.
- (2) To check if the data is erased, press MEM button, no data should show up on the screen.

4. Changing batteries:

When battery low symbol  shows up on the display, the battery is weak. Take out all four batteries and put in 4 new "AA" Alkaline batteries.

***Attention!**

- All batteries used must be the same type. Do not mix alkaline, standard (carbon-zinc) or rechargeable (cadmium) batteries.**
- Do not mix old and new batteries.**
- When you change the batteries, all stored data will disappear.**

MAINTENANCE

1. Use a piece of cloth with water or a mild cleansing agent to clean the case and then use a piece of dry cloth to wipe it dry. Use a piece of dry cloth to wipe the cuff when it is dirty.
2. Do not use alcohol or any strong cleansers when cleaning.
3. If the unit is not going to be used for a long time, remove the batteries. (Leaking of battery liquid can cause serious damage to the unit).

SAFE-KEEPING

1. Always keep the unit in the carrying case after its use.
2. Do not put the item directly in sunlight, in high temperatures, or humid and dusty places.
3. Do not store in extremely low (less than 20° F / -7° C) or high (more than 110° F / 43° C) temperatures.

TO PREVENT MALFUNCTION

1. Do not in any way twist the arm cuff
2. Do not start the start measurement when the arm cuff is not wrapped around the arm.
3. Do not attempt to disassemble or change any parts of the monitor including the cuff.
4. Do not drop the product or cause any strong impact.

WARNING

1. The cuff size is suitable for 9 ~ 13 inches / 23 ~ 33 cm adult only (HM-30-OB : 33 ~ 43 cm / 13 ~ 17 in).
2. The device is not supposed to be used if your arm has any wound or injury.
3. If the cuff continues to inflate without stopping, open the cuff immediately.
4. Substituting a component different from that supplied might result in measurement error.

WARRANTY AND SERVICE

This unit is warranted against defects in manufacturing for 2 years from date of purchase. Except in cases of abuse, Prestige Medical will repair defective sets during the warranty period. Batteries or damage from batteries is not covered by the warranty. You may return the defective unit to us - postage paid - with \$5 for return shipping and handling and proof of purchase. Except as limited by law, the liability of Prestige Medical includes no incidental or consequential damages.

Prestige Medical
8600 Wilbur Avenue
Northridge, California 91324
www.prestigemedical.com

NCD Medical
Dermotstown, Naul
Co. Dublin, Ireland

DESCRIPTION OF DISPLAY SYMBOLS

Display mark	Condition/Cause	Corrective action
 Pulse symbol	Flashes on the LCD screen when pulse is detected during measurement.	Measurements in progress. Do not move or talk.
 Low battery symbol	Appears when the batteries life is extremely low.	Replace all four batteries with new ones. Insert the batteries in correct polarities.
 Measuring error symbol	Appears when an accurate blood pressure could not be obtained.	Press "START/STOP" button again and re-measure. Check that the cuff is wrapped properly around the arm as in the instructions. Check for kinks in the air tube. Do not talk or move during the measurement. Check if your posture is correct.

SPECIFICATIONS

Range of measurement	Pressure 0 ~ 300 mmHg, Pulse 40 ~ 199 beats/minute
Accuracy	Pressure +/- 3mmHg, Pulse +/- 5% Max.
Inflation	Automatic
Exhaust	Automatic exhaust valve
Display	Systolic, Diastolic, Pulse rate
Sets of memory	99 sets
Cuff size - HM-30	Arm circumference approx. 23 ~ 33 cm / 9 ~ 13 inch
Cuff size - HM-30-OB	Arm circumference approx. 33 ~ 43 cm / 13 ~ 17 inch
Ambient Operation	50° F ~ 104° F (10° C ~ 40° C) less than 85% R.H.
Transport/Storage	-13° F ~ 158° F (-25° C ~ 70° C) less than 85% R.H.
Unit weight	Approx. 265 g with batteries
Power Supply	AA (1.5 V) Alkaline Battery x 4
Battery life	300 measurements (twice a day measurement)
Auto power off	Whenever not used for 1 minute
Accessories	4 batteries, arm cuff with tube, instruction manual, pouch, gift box.

NOTE

This Blood Pressure Monitor complies with the EC Directive and bears the CE mark "CE 0120".

The quality of the device has been verified and is in line with the provisions of the EC council directive 93/42/EEC (EN IEC60601-1 General requirements for safety, EN IEC60601-1-2:2001 Electromagnetic compatibility-Requirements and tests,) dated 14 June 1993 concerning medical devices and the EN performance standards as following:

EN 1060-1 Non-invasive blood pressure measuring equipment. General requirements
(Dec. 1995)

EN 1060-3 Non-invasive blood pressure measuring equipment. Supplementary requirements for electromechanical blood pressure measuring systems. To ensure accurate measurements, it is recommended that calibration be completed every two years.

NOTE

Important!/Caution/Note!
Read the operating instructions.

Consult instructions thoroughly before use.

BF

Classification:

- Internally powered equipment
 - BF type applied part
 - IPX0
 - Not suitable for use in presence of flammable anaesthetic mixture with air or with Oxygen or nitrous oxide
 - Continuous operation with short-time loading
-

To avoid inaccurate results caused by electromagnetic interference between electrical and electronic equipments, do not use the device near a cell phone or microwave oven.

Discard the used product to the recycling collection point according to local regulations.

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